




TESTING MADE EASY

Testing your blood sugar

1 Once you have loaded the 10-test disc (see reverse side), hold the meter in your hand or on a flat surface. Pull out **(1a)** and then push back in **(1b)** the meter handle to expose a test strip. The meter will automatically turn on, and the “apply blood” prompt will appear on the screen. The meter is now ready to test.

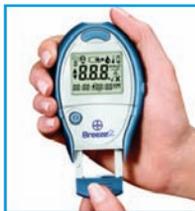

1a




1b

2 Touch the tip of the test strip to the blood drop. Hold in place **(2a)** until the meter beeps and countdown begins from 5 to 1. The result is displayed **(2b)** after 5 seconds and is stored in the meter automatically for future reference.



2a

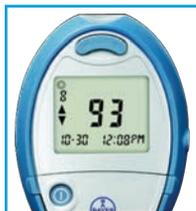

2b

3 To discard the test strip, hold the meter pointing down over a waste container and press the blue release button at the top. Then turn the meter off.



3

For more information on Bayer products and services, visit www.BayerDiabetes.com/us





TESTING MADE EASY

Loading the BREEZE[®]2 Test Disc

- 1 To open the meter, hold it with the screen facing down. Press in on the gray latch on the top edge of the meter, and then pull up.
- 2 Insert a new 10-test disc, bumpy side up, keeping the meter flat. Gently snap the meter shut.



1




2

Note: When you push the “On/Off” button, the screen will display how many test strips remain in the disc in the top left-hand corner.



The BREEZE[®]2 Meter and 10-test disc received the Ease-of-Use Commendation from the Arthritis Foundation.

Function Buttons

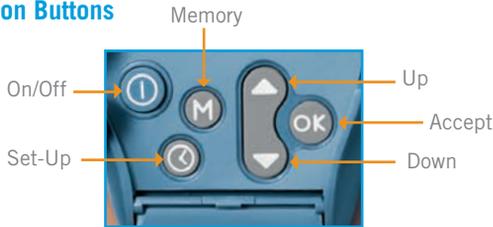

If you have any questions, please refer to the **User Guide for the BREEZE[®]2 Meter.**

Bayer Customer Service provides personal assistance when you need it. **Call 1-800-348-8100.**

Note: BREEZE[®]2 Test Strips do not work with the original BREEZE[®] Meter; use only with the BREEZE[®]2 meter.

